

"be free" Odor Eliminator and Pod Instructional Tips




"be free" Products
Odor Eliminator and Pods



Gather your supplies as you normally do prior to changing your pouch.




Remove the backing from your barrier.

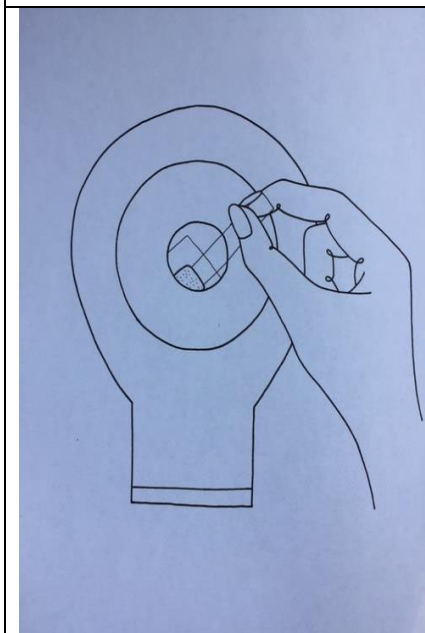

Insert "be free" pod through barrier opening into the pouch.



Press the tail of pod onto the adhesive barrier until secured.

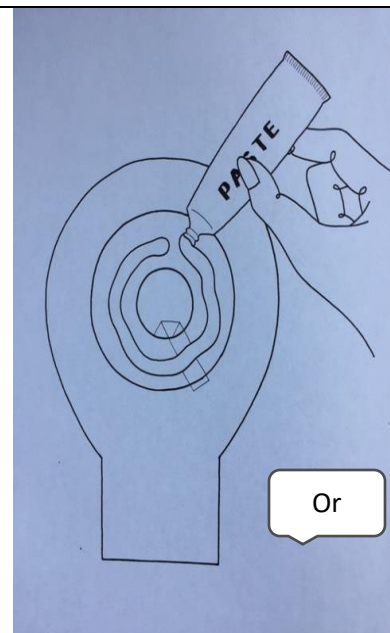


Apply your paste...


“be free” Odor Eliminator and Pod Instructional Tips



Apply barrier ring as you normally do directly over the pod's plastic tail to secure it in place.

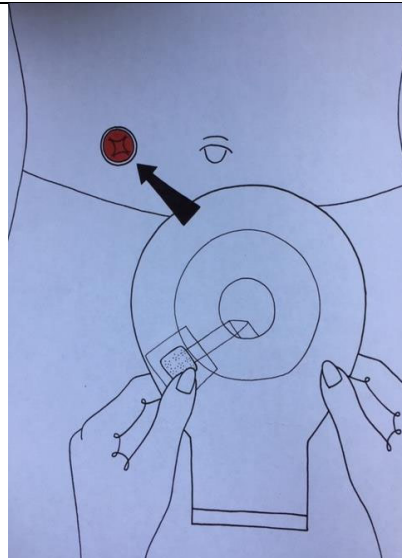


Apply your pouch (with pod inside) over your stoma site.

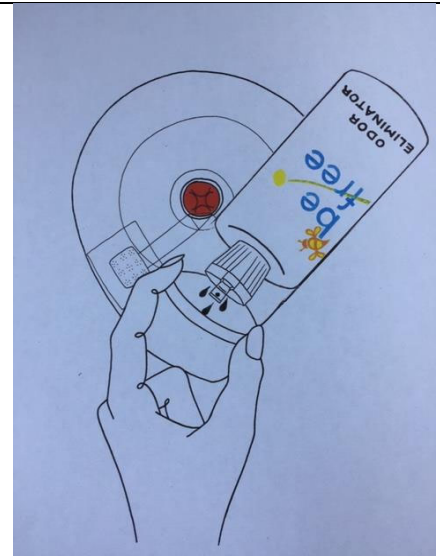


Squirt “be free” odor eliminator into the pouch.



Lift pouch tail so liquid deodorant can reach the “be free” pod. Squeeze the pod to absorb the liquid. Some liquid will be left in the pouch.

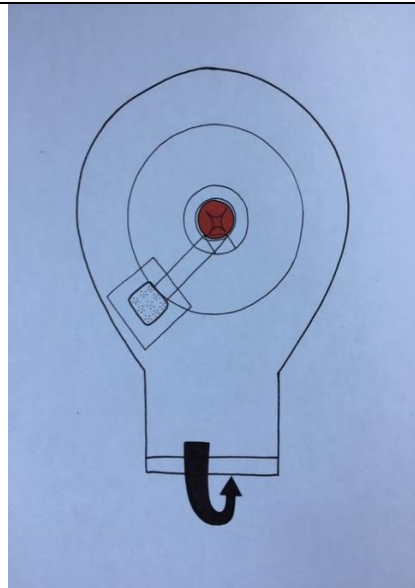

Close tail of pouch.

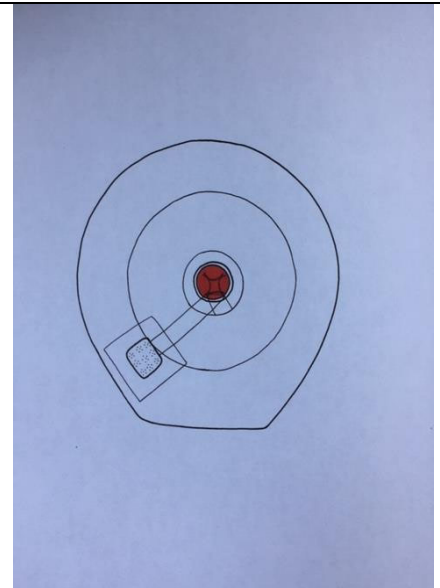

“be free” from odor!! Repeat deodorant application two times a day or as needed. Change your pouch twice a week or as you normally do.

INGREDIENTS: Purified Water, Sodium Citrate, Coco Betaine, Zinc Ricinoleate, Citric Acid, Propylparaben, Methylparaben, Diazolidinyl Urea, Acemannan Hydrogel, polyurethane foam.

CAUTION: Do not put silica gel pack in ostomy.